


Angled Screw Channel Restoration Delivery Protocol



Tools needed:

- Torque wrench *(implant brand specific)

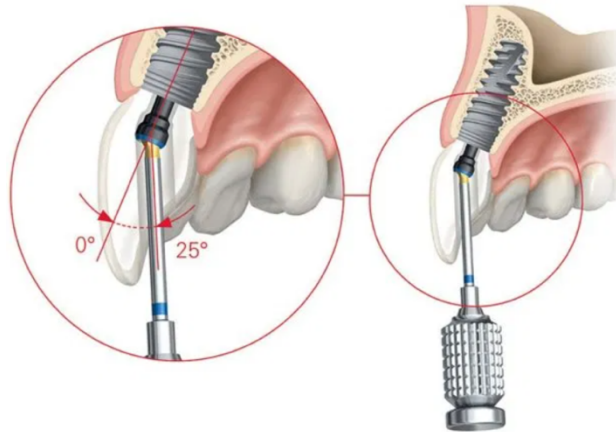

- Angled screw channel (ASC) driver *(implant brand specific)



- Colored Teflon tape - To easily identify ASC screw from non-ASC screw when retrieval is needed.

Instruction:

1. Engage ASC driver with the ASC screw.
2. Screw down and torque the restoration at manufacturer's suggested torque value *(implant brand specific). [Torque value for compatible ASC is 30 N-cm]
3. Insert colored Teflon tape in screw channel prior to sealing access hole with composite to prevent composite from covering the screw head.



*Torque wrench, angled screw channel driver, and torque value are implant brand specific. Consult with your implant system representative for proper tools and delivery of angled screw channel abutment.